SUPERLINE-A

Instruction For Use

Dána


SUPERLINE-A infusion set

For Use Only With DANA Insulin Pumps

English

Instructions for Use

The SUPERLINE-A is an infusion set with a stainless steel needle. A needle is inserted through the skin, at 90° with the skin, into the subcutaneous tissue. The plaster of the SUPERLINE-A holds the needle in place.

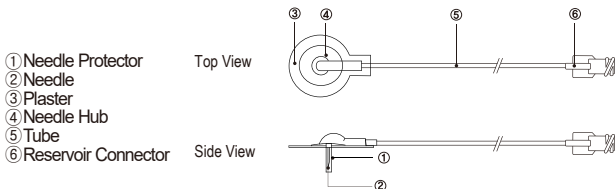① Needle Protector

② Needle

③ Plaster

④ Needle Hub

⑤ Tube

⑥ Reservoir Connector

Top View

Side View

| | | | |
|--------------------|-------|-------|-------|
| Needle Length (mm) | 6.5 | 8.0 | 4.5 |
| Tube Length (mm) | 550 | | |
| Model Name | SU001 | SU002 | SU003 |

General Precautions for All Infusion Sets

1. Replace the reservoir and the infusion set every 3 days as recommended by your health care professional.
2. The reservoir and infusion set are intended for single use only. Do not reuse either the reservoir or infusion set. Reuse can cause serious illnesses (Septicemia, HBV, HVC, etc.) from bacterial infection.
3. Do not use the reservoir or infusion set if the package has been damaged, inadvertently opened or wet.
4. The shelf life of the reservoir and the infusion set is 3 years. Please check expiration dates regularly and dispose of any expired accessories.
5. Do not disconnect from your pump for more than one hour without discussing with your healthcare professional on how you should compensate for missed insulin. Carefully monitor blood glucose levels during periods of disconnection and after reconnection.
6. Check frequently to make sure the needle remains firmly in place as you may not feel pain if it pulls out. The needle must always be completely inserted to receive the full amount of medication.
7. During use, if adhesive strength is unstable or want to fix the tape more firmly, may be recommended using overpatch.
8. If the infusion site becomes inflamed, replace the set, and use a new site until the first site has healed.
9. Replace the infusion set if the tape becomes loose, or if the needle becomes fully or partially dislodged from the skin.
10. Avoid contact with acetone or alcohol, such disinfection products may result in abnormalities or damage the plastics.
11. Store infusion set in a cool, dry location at room temperature.
12. The SUPERLINE-A is to be used by patients who have received training from a certified diabetes educator and/or insulin pump trainer and by advice from a physician.

Intended use for SUPERLINE-A

SUPERLINE-A infusion sets are intended for the subcutaneous infusion of insulin from a SOOIL pump.

Contraindications

The SUPERLINE-A can only be use with DANA insulin pump, so it should not use for any purpose other than to use an insulin pump. The infusion set is neither intended nor indicated for use with blood.

Adverse Event Reporting

If you experience any malfunction or deterioration in the characteristics and/or performance of a device which, directly or indirectly might lead to the death of a patient, or to a serious deterioration in their state of health, report to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

The SUPERLINE-A Infusion Set Instruction

1. Prior to Inserting the SUPERLINE-A Infusion Set

- 1) Wash your hands thoroughly before applying the SUPERLINE-A.
- 2) Take out the infusion set from its package. Keep the Reservoir Connector sterile from disinfectant, perfume, deodorant, or repellent.
- 3) Screw the Reservoir Connector counterclockwise into the Reservoir Cap of the DANA insulin pump until it is firmly in place.

2. Inserting the SUPERLINE-A Infusion Set

- 1) Choose an insertion site on your abdomen at least 5 cm away from the navel. Clean the site with an alcohol swab and let it dry (fig. 1)
- 2) Prime the infusion set by following the DANA Insulin pump User Manual.
- 3) Remove the plaster backing paper and the needle protector by gently pulling it. Check the needle and replace if damaged (fig. 2)
- 4) Take a big pinch of skin between your thumb and index finger and hold it. Insert the needle firmly into the pinched skin at a 90 (fig. 3)
- 5) Secure the needle with the plaster and tape the tube to the body with adhesive tape 2~5 cm away from the insertion site (fig. 4)

- Notice**
- Try to avoid any recent insertion sites, scars, scar tissue or bruising.
 - Remember to prime at least 10 units when you replace the reservoir and infusion set to remove for safe insulin delivery.
 - When priming the infusion set, hold the set with the needle pointing down to ensure that medication does not come in contact with the adhesive backing paper.

3. Removing the SUPERLINE-A Infusion Set

- 1) Place the pump in SUSPEND
- 2) Carefully remove the adhesive tape from the insertion site and tube.
- 3) Remove the Needle in the same manner as if you were inserting it. Use gauze to apply light pressure to the infusion site.
- 4) Remove the infusion set from the pump by turning the Reservoir Connector clockwise and pulling upwards. Dispose of the used infusion set properly.

Additional information for The SUPERLINE-A Infusion Set

Insulin for use with SUPERLINE-A Infusion Set

The infusion sets have been tested using the insulins listed below

| Insulin type | Product Name | Product License |
|--------------|--------------|-----------------|
| LISPRO | Humalog | EMA/477399/2014 |
| ASPART | Novorapid | EMA/632418/2016 |

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DE IME-DC GmbH
FR MEDLIFE SAS
RU AVANTMED LTD.