# DANA 3cc Syringe

Instruction For Use


# DANA 3cc Syringe

For Use Only With DANA Insulin Pumps

English

Instructions for Use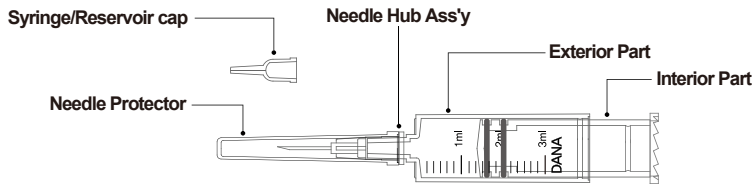

## General Precautions

1. Replace the reservoir every 3 days as recommended by your health care professional.
2. The reservoir are intended for single use only. Do not reuse either the reservoir or infusion set. Reuse can cause serious illnesses (Septicemia, HBV, HVC, etc.) from bacterial infection.
3. Do not use the reservoir if the package has been damaged, inadvertently opened or wet.
4. The shelf life of the reservoir is 3 years. Please check expiration dates regularly and dispose of any expired.
5. Store reservoir in a cool, dry location at room temperature.
6. The DANA 3cc Syringe is to be used by patients who have received training from a certified diabetes educator and/or insulin pump trainer and by advice from a physician.

## Intended User

DANA 3cc Syringe is a disposable reservoir to be set up inside of DANA Insulin Infusion Pumps. All the users of DANA Insulin Infusion Pumps and DANA Infusion Sets are the intended users for DANA 3cc Syringe, because these pumps are designed to store insulin only in DANA 3cc Syringe and DANA Infusion Sets must be connected only to this syringe to deliver insulin into the subcutaneous tissue.

## Intended Purpose

DANA 3cc Syringe, a disposable reservoir to be set up inside of DANA Insulin Infusion Pumps, is intended to store insulin for up to 72 hours and to release insulin through DANA Infusion Sets at the rate set by the insulin pump. Briefly, one side of this syringe is connected to the motor of the insulin pump through the linking screw which rotates to generate a linear movement of the plunger of the syringe to release insulin. The other side of this syringe is connected to the cannula of DANA Infusion Sets through which insulin is eventually delivered into the subcutaneous tissue.

## Indication for Use

Youth and adults on DANA Insulin Infusion Pumps with type 1 diabetes and individuals on DANA Insulin Infusion Pumps with any type of insulin-deficient diabetes, including type 2 diabetes, who are capable of using DANA 3cc Syringe safely.

## Contraindications

1. Patients using other insulin pumps than DANA Insulin Infusion Pumps.
2. Patients who are not able to change DANA 3cc Syringe as the instruction manual guides for avoiding contamination, if a helper is not provided.

## Additional information for the 3cc Syringe

### Insulin for use with 3cc Syringe

The 3cc Syringe have been tested using the insulins listed below

Insulin type	Product Name	Product License
LISPRO	Humalog	EMA/477399/2014
ASPART	Novorapid	EMA/632418/2016

## Preparation

Loading and refilling the Insulin Pump with insulin is a technical process which involves medication (insulin) and sterile components. It is recommended that you:

1. Retrieve the insulin vial from the refrigerator and let it warm up to room temperature.
2. SUSPEND the insulin pump and remove the old Infusion Set.
3. Remove the old reservoir.
4. Place all necessary components on a clean dry surface with good lighting.
  - DANA Insulin Pump
  - Fast acting insulin
  - DANA Reservoir (3ml)
  - DANA Infusion Set
  - 1 x Alcohol swab
  - Pump linking screw
  - DANA Auto Setter or DANA Easy Setter
5. Wash and dry your hands before opening sterile packets.
6. Make sure Off the SUSPEND on the insulin pump before REFILL procedure.


**Caution** Using insulin directly from the fridge can cause micro air bubbles in the reservoir and tubing. Allow the insulin to reach room temperature before starting the refill process. When filling the reservoir, take care to remove all air bubbles.

**Notice** Tolerance on the graduated capacity is  $\pm 4\%$ .


## The DANA 3cc Syringe Instruction


- ① Remove the round cap at the end of the reservoir plunger and discard. A small white 'Reservoir Cap' will fall out of the plunger – keep this cap for later use.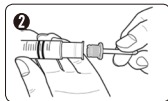

- ② Insert and fit the blue plastic component of the linking screw into the cavity of the plunger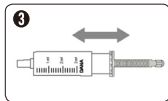


- ③ Pull back on the plunger to the line marked with the 3ml. Push up and 3-4 times to lubricate the reservoir.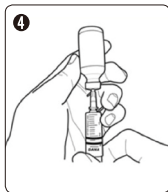


- ④ Wipe the lid of the insulin vial with an alcohol swab. Carefully remove the clear 'Needle Protector' and draw out the desired amount of insulin. If refilling from a 10ml vial you may need to inject air to match the volume of insulin you are attempting to withdraw.

**Notice** When refilling from a 10ml Insulin vial, pull the plunger down until you are at the desired amount of insulin needed. Insert the needle into the insulin vial and inject the air from the reservoir into the vial. Draw out the desired amount of insulin.

\*Suggested fill amount formula: (the usual daily requirement x 3days)+extra 40u.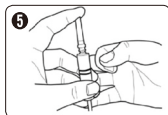

- ⑤ Place the clear 'Needle protector' back on the needle. Remove any bubbles from the inside of the reservoir by gently tapping on the sides, making the bubbles rise to the top of the reservoir. Gently push on the plunger to remove bubbles at the top.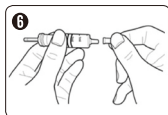

- ⑥ Remove the needle from the reservoir and cover with the small white 'Reservoir Cap'. Dispose of the needle properly.


- ⑦ Adjust the length of the linking screw with Easy Setter or Auto Setter.

**Notice** You are required to use the Auto Setter or Easy Setter before inserting Reservoir into the insulin pump. Refer to DANA insulin pump instruction for use for the detailed methods to use them.

- ⑧ Insert the reservoir with linking screw into the Insulin Pump like as shown. When inserting the reservoir, rotate the reservoir back and forth until the notch fittings on the side of the reservoir slide into place. Gently let the reservoir and linking screw fall into place.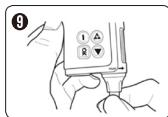

- ⑨ Screw (counterclockwise) the Insulin Pump 'Reservoir Cap' in place and remove the small white 'Reservoir Cap' from the end of the reservoir.

## Adverse Event Reporting

If you experience any malfunction or deterioration in the characteristics and/or performance of a device which, directly or indirectly might lead to the death of a patient, or to a serious deterioration in their state of health, report to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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Global MED LTD
**PL****IME-DC GmbH**
**TR****İME DC Sağlık Ürünleri San. ve Tic. Ltd. Şti.**