Easy Release

Instruction For Use

Dana


Easy Release infusion set

For Use Only With DANA Insulin Pumps

English

Instructions for Use

The Easy Release Infusion Set is a stainless steel needle set. A needle is inserted at 90° into the skin. The adhesive of the Easy Release allows the needle to be fixed firmly in place.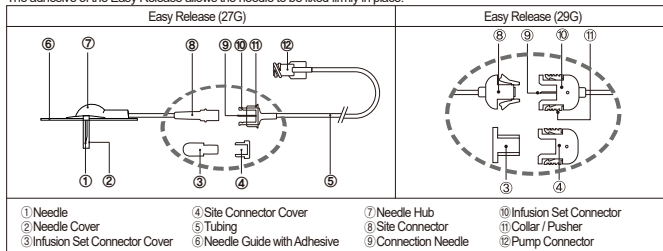

The insulin residual volume of head part (from ① Needle to ⑧ Site Connector) is (7mm needle: 1.6 units), (9mm needle: 1.8units)

Needle Length (mm)	Total Tube Length (mm)					
	100	600	700	1,100		
ER	27G	4.5	SU303	SU317	SU315	SU316
		7	SU301	SU318	SU313	SU311
		9	SU302	SU319	SU314	SU312
	29G	4.5	ER0145	ER0645	ER0745	ER1145
		7	ER0170	ER0670	ER0770	ER1170
		9	ER0190	ER0690	ER0790	ER1190

General Precautions for All Infusion Sets

1. Replace the reservoir and the infusion set every 3 days as recommended by your health care professional.
2. The reservoir and infusion set are intended for single use only. Do not reuse either the reservoir or infusion set. Reuse can cause serious illnesses (Septicemia, HBV, HVC, etc.) from bacterial infection.
3. Do not use the reservoir or infusion set if the package has been damaged, inadvertently opened or wet.
4. The shelf life of the reservoir and the infusion set is 3 years. Please check expiration dates regularly and dispose of any expired accessories.
5. Do not disconnect from your pump for more than one hour without discussing with your healthcare professional on how you should compensate for missed insulin. Carefully monitor blood glucose levels during periods of disconnection and after reconnection.
6. Check frequently to make sure the needle remains firmly in place as you may not feel pain if it pulls out. The needle must always be completely inserted to receive the full amount of medication.
7. During use, if adhesive strength is unstable or want to fix the tape more firmly, may be recommended using overpatch.
8. If the infusion site becomes inflamed, replace the set, and use a new site until the first site has healed.
9. Replace the infusion set if the tape becomes loose, or if the needle becomes fully or partially dislodged from the skin.
10. Avoid contact with acetone or alcohol, such disinfection products may result in abnormalities or damage the plastics.
11. Store infusion set in a cool, dry location at room temperature.
12. The Easy Release is to be used by patients who have received training from a certified diabetes educator and/or insulin pump trainer and by advice from a physician.

Intended User

Easy Release is a disposable infusion set through which insulin is delivered into the subcutaneous tissue, exclusively while connected to DANA Insulin Infusion Pumps. All the users of DANA Insulin Infusion Pumps are the intended users for Easy Release, because this pump is designed to deliver insulin only through DANA Infusion sets.

Intended Purpose


Easy Release is intended for the delivery of insulin into the subcutaneous tissue, exclusively while connected to DANA Insulin Infusion Pump. This infusion set is neither intended nor indicated for the delivery of anything other than insulin.

Indication for Use

Youth and adults on Diabecare DANA-i Insulin Pump with type 1 diabetes and individuals on DANA Insulin Infusion Pump with any type of insulin-deficient diabetes, including type 2 diabetes, who are capable of using Easy Release safely.

Contraindications

1. Patients using other insulin pumps than DANA Insulin Infusion Pump.
2. Patients suffering from intractable allergic dermatitis to the adhesive, a material used to attach the needle onto the skin, which belongs to Easy Release.
3. Patients who are not able to use Easy Release as the instruction manual guides for avoiding contamination, if a helper is not provided.

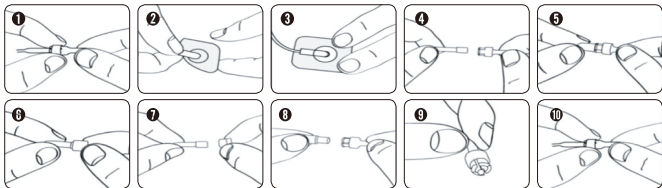Easy Release Infusion Set Instruction

1. Prior to inserting Easy Release Infusion Set.

- 1) Wash your hands before handling the infusion set.
- 2) Take out the Easy Release Infusion Set from its package. Be careful not to touch the connector to any object that would breach its sterility.
- 3) Screw the Pump Connector counterclockwise into the reservoir compartment unit it is firmly in place.
- 4) Push the Set Connector firmly into the Site Connector and pull the Collar back to secure the connection (fig.1)

2. Inserting the Easy Release Infusion Set

- 1) Choose an insertion site on your abdomen at least 5 cm away from the navel. Clean the site with an alcohol swab and let it dry.
- 2) Prime the Easy Release infusion set by following instructions. You can get more information on prime procedure from the User Manual of DANA Insulin pump.
- 3) Carefully remove the needle's protective cover, and the paper backing from the adhesive dressing.
- 4) Grab the tissue of insertion site with the thumb and finger of your other hand. Push the needle firmly into the tissue (fig.2)
- 5) Secure the needle with the adhesive dressing (fig.3)
- 6) Tape the Easy Release infusion set to the body with adhesive tape 2~5 cm away from the insertion site.

Notice

- Try to avoid any recent insertion sites, scars, scar tissue or bruising.
- Remember to prime at least 15~20 units (depending on tube length) when you replace the reservoir and infusion set to remove for safe insulin delivery.
- When priming the infusion set, hold the set with the needle pointing down to ensure that medication does not come in contact with the adhesive backing paper.

3. Disconnecting the Easy Release Infusion Set

- 1) Place the pump in SUSPEND.
- 2) Push the Ring forward and pull the Infusion Set Connector out of the Site Connector (fig.4).
- 3) Push the Set Connector Cover onto the Connecting Needle (fig.5). Make sure that this is done without bending the Connecting Needle. This prevents insulin from dripping out at the release site and keeps the Connecting Needle clean.
- 4) Insert the Site Connector Cover into the Site Connector (fig.6).
- 5) Put your pump in a clean and safe place until you are ready to reconnect.

4. Reconnecting the Easy Release Infusion Set

- 1) Wipe the tip of the Site Connector with an alcohol swab after removing the Site Connector Cover (fig. 7).
- 2) Remove the Set Connector Cover from the Set Connector (fig.8).
- 3) Take pump out of SUSPEND.
- 4) Before reconnecting, bolus 0.5~1 unit of insulin and watch to ensure that a drop is visible from the Connection Needle tip (fig.9)
- 5) Gently slide the Set Connector into the Site Connector, and pull the Ring back to secure the connection (fig.10).

5. Removing the Easy Release Infusion Set

- 1) Place the pump in SUSPEND
- 2) Carefully remove the adhesive tape from the Tubing and needle insertion site.
- 3) Remove needle in the same manner as if you were inserting it. Use gauze to apply light pressure to the insertion site.
- 4) Remove the infusion set from the pump by turning the Reservoir Connector clockwise and pulling upwards. Discard the used infusion set properly.

Adverse Event Reporting

If you experience any malfunction or deterioration in the characteristics and/or performance of a device which, directly or indirectly might lead to the death of a patient, or to a serious deterioration in their state of health, report to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Additional information for The Easy Release Infusion Set

Insulin for use with Easy Release Infusion Set

The infusion sets has been tested using the insulins listed below.

Insulin type	Product Name	Product License
LISPRO	Humalog	EMA/477399/2014
ASPART	Novorapid	EMA/632418/2016

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Global MED LTD
PL IME-DC GmbH
TR İME DC Sağlık Ürünleri San. ve Tic. Ltd. Şti.