# Soft Release-ST

Instruction For Use

Dána

# Soft-Release-ST infusion set


## For Use Only With DANA Insulin Pumps

English

Instructions for Use

The Soft-Release-ST is a soft cannula infusion set. It may be inserted at any angle between 15° and 30°, depending on your body type, comfort and instructions from your healthcare professional. The needle is removed after insertion and only the cannula remains under the skin.

- ① Teflon (Soft Cannula)
- ② Needle Guide with Adhesive
- ③ Site Connector
- ④ Pump Connector
- ⑤ Tubing
- ⑥ Connection Needle
- ⑦ Infusion Set Connector
- ⑧ Spring Clips
- ⑨ Infusion Set Connector Cover
- ⑩ Site Connector Cover
- ⑪ Needle Hub
- ⑫ Introducer Needle
- ⑬ Needle Cover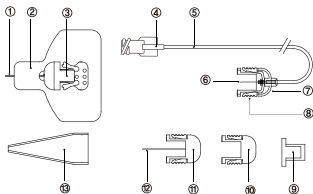

Model Names	SR102	SR112
Needle Length (mm)		19
Total Tube Length (mm)	700	1,100

### General Precautions for All Infusion Sets

1. Replace the reservoir and the infusion set every 3 days as recommended by your health care professional.
2. The reservoir and infusion set are intended for single use only. Do not reuse either the reservoir or infusion set. Reuse can cause serious illnesses (Septicemia, HBV, HVC, etc.) from bacterial infection.
3. Do not use the reservoir or infusion set if the package has been damaged, inadvertently opened or wet.
4. The shelf life of the reservoir and the infusion set is 3 years. Please check expiration dates regularly and dispose of any expired accessories.
5. Do not disconnect from your pump for more than one hour without discussing with your healthcare professional on how you should compensate for missed insulin. Carefully monitor blood glucose levels during periods of disconnection and after reconnection.
6. Check frequently to make sure the needle remains firmly in place as you may not feel pain if it pulls out. The needle must always be completely inserted to receive the full amount of medication.
7. During use, if adhesive strength is unstable or want to fix the tape more firmly, may be recommended using overpatch.
8. If the infusion site becomes inflamed, replace the set, and use a new site until the first site has healed.
9. Replace the infusion set if the tape becomes loose, or if the needle becomes fully or partially dislodged from the skin.
10. Avoid contact with acetone or alcohol, such disinfection products may result in abnormalities or damage the plastics.
11. Store infusion set in a cool, dry location at room temperature.
12. The Soft-Release-ST is to be used by patients who have received training from a certified diabetes educator and/or insulin pump trainer and by advice from a physician.

### Intended use for Soft-Release-ST


The Soft-Release-ST is intended for the subcutaneous delivery of insulin for the treatment of diabetes mellitus.

### Contraindications

The Soft-Release-ST can only be use with DANA insulin pump, so it should not use for any purpose other than to use an insulin pump. The infusion set is neither intended nor indicated for use with blood.

### Adverse Event Reporting

If you experience any malfunction or deterioration in the characteristics and/or performance of a device which, directly or indirectly might lead to the death of a patient, or to a serious deterioration in their state of health, report to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

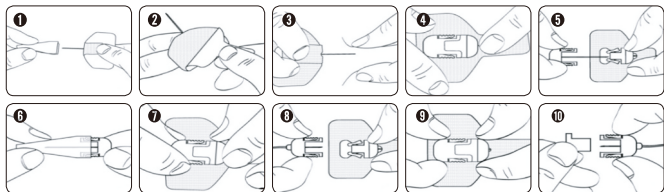## The Soft-Release-ST Infusion Set Instruction

### 1. Prior to Inserting the Soft-Release-ST Infusion Set

- 1) Wash your hands thoroughly before applying the Soft-Release-ST.
- 2) Take out the Infusion Set from its package. Keep the Reservoir Connector sterile from disinfectant, perfume, deodorant, or repellent.
- 3) Screw the Reservoir Connector counterclockwise into the reservoir compartment until it is firmly in place.

### 2. Inserting the Soft-Release-ST Infusion Set

Choose an insertion site on your abdomen at least 5 cm away from the navel. Clean the site with an alcohol swab and let it dry.

#### (1) Inserting by Manual

- 1) Hold the Cannula Hub and remove the needle protector. Keep the needle protector for proper disposal of the insertion needle (fig.1)
- 2) Remove the bottom portion of the plaster backing paper from the plaster (fig.2)
- 3) Take a big pinch of skin between your thumb and index finger and hold it. Insert the needle firmly into the pinched skin at a 15-30° with the skin (fig.3). Watch the IV Cannula Hub Ass'y during insertion to ensure that it goes in smoothly without kinking or crimping.
- 4) Remove the top portion of the plaster backing paper and attach the plaster to the skin to hold the cannula in place (fig.4)
- 5) To remove the Insertion needle, press Spring Clips of the Needle Hub and pull the needle straight out (fig.5)
- 6) Dispose of the Insertion Needle in a safe manner by using the Needle Protector (fig.6)

#### (2) Inserting by Inserter-ST

- 1) Push down the Inserter-ST's holder until you hear a click and place the safety button to the lock position.
- 2) Prepare the Soft-Release-ST and hold the needle cover with your hand and put Soft-Release ST into the cannula holder of the Inserter-ST.
- 3) Peel off the part of plaster backing paper of the Soft-Release ST and place the inserter-ST on the expected insertion site considering insertion degree. (15-30°degrees are recommendable). Unlock the safety button then push the trigger to insert the cannula with the insertion needle.
- 4) Press the plaster on to the skin and remove the Inserter-ST. Then, remove the needle hub and dispose of the needle with the needle protector.
- 5) Peel off another part of the plaster backing paper lifting cannula slightly and holding plaster then firmly attach the entire plaster to skin.

#### (3) After the insertion

- 1) Prime the infusion set by following the DANA Insulin pump User Manual.
- 2) Gently slide the Set Connector into the Cannula hub until you hear a click (fig.7)
- 3) Tape the tube to the body with adhesive tape 2-5 cm away from the insertion site.

#### Notice

- Try to avoid any recent insertion sites, scars, scar tissue or bruising.
- Remember to prime at least 15-20 units (depending on tube length) when you replace the reservoir and infusion set to remove for safe insulin delivery.
- When priming the infusion set, hold the set with the needle pointing down to ensure that medication does not come in contact with the adhesive backing paper.

### 3. Disconnecting the Soft-Release-ST Infusion Set

- 1) Place the pump in SUSPEND.
- 2) Grasp each side of the Set Connector, pressing gently on the Spring Clips. Pull the Set Connector out of the Cannula Hub (fig.8)
- 3) Push the Set Connector Cover onto the Connecting Needle (fig.10). Make sure that this is done without bending the Connecting Needle. This prevents insulin from dripping out at the release site and keeps the Connecting Needle clean.
- 4) Insert the Site Connector Cover into the Site Connector (fig.10)
- 5) Put your pump in a clean and safe place until you are ready to reconnect.

### 4. Reconnecting the Soft-Release-ST Infusion Set

- 1) Wipe the tip of the Site Connector with an alcohol swab after removing the Site Connector Cover (fig.7)
- 2) Remove the Set Connector Cover from the Set Connector (fig.10).
- 3) Take the pump out of SUSPEND.
- 4) Before reconnecting, bolus 0.5-1 unit of insulin and watch to ensure that a drop is visible from Connecting Needle tip.
- 5) Gently slide the Set Connector into the Site Connector until you hear a click (fig.9)

### 5. Removing the Soft-Release-ST Infusion Set

- 1) Place the pump in SUSPEND.
- 2) Carefully remove the adhesive tape from the tube and insertion site.
- 3) Grab the Needle Hub in the same manner as if you were inserting it. Use your other hand to apply slight pressure on the skin around the insertion area. Gently pull out the cannula.
- 4) Remove the infusion set from the pump by turning the Reservoir Connector clockwise and pulling upwards.
- 5) Dispose of the used infusion set properly.

## Additional information for The Soft-Release-ST Infusion Set

### Insulin for use with Soft-Release-ST Infusion Set

The infusion sets have been tested using the insulins listed below

Insulin type	Product Name	Product License
LISPRO	Humalog	EMA/477399/2014
ASPART	Novorapid	EMA/632418/2016

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Global MED LTD
**NL IME-DC GmbH**
**PL IME-DC GmbH**